
Creation of OSCE contents and PBL case studies ·
Log management cloud system

iPax Studio

Create case studies and quiz questions, Collectively manage grades and logs
A set of user IDs and management functions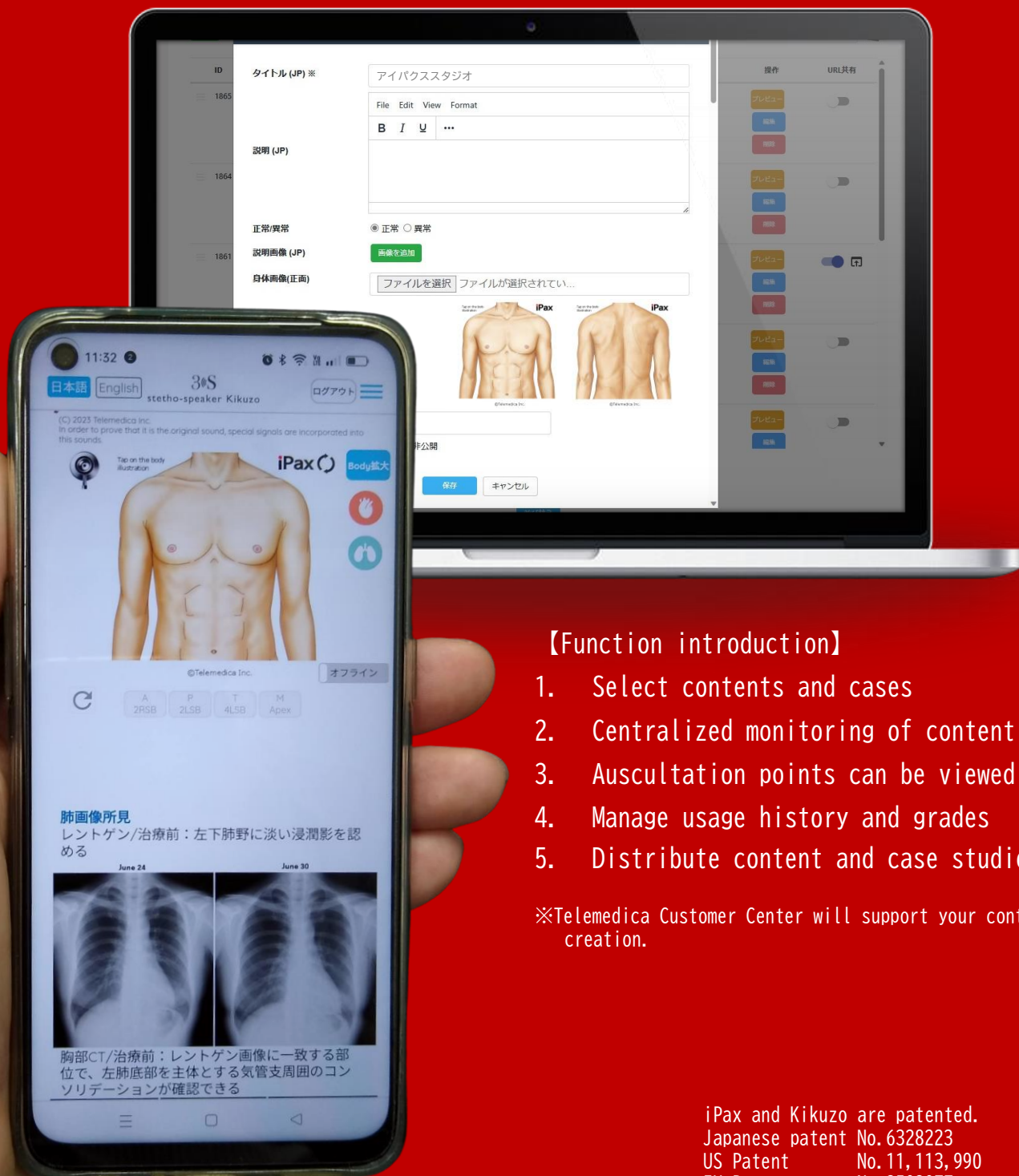

【Function introduction】

1. Select contents and cases
2. Centralized monitoring of content
3. Auscultation points can be viewed
4. Manage usage history and grades
5. Distribute content and case studies

※Teledemica Customer Center will support your content creation.

iPax and Kikuzo are patented.
Japanese patent No. 6328223
US Patent No. 11, 113, 990
EU Patent No. 3503077
iPax Studio patent pending

1 Select cases

Search for the case you want to use from the iPax Library on the left menu of the management screen. Turn on the “Share URL” button on the right side of the displayed case, and press the arrow mark further right of it to display a pop-up screen.

Administrator screen

The screenshot shows the iPax Library interface. On the left is a navigation menu with categories like Home, Analytics, Exam Management, Quiz Management, Question Management, Library Management, User Management, and Live Monitoring. The main area displays a table of cases. The third case, 'SAMPLE Case', has its 'Share URL' button highlighted with a red box. Below the table is a pagination control and a 'Sort libraries' button.

ID	Case Name	Category	Status	Access	Date	Admin	Actions
1865	AS ver.23 (Non pulse, 1min)	Heart Sound	Abnormal	Public	2023-06-30 2:32:134	Admin	Preview, Edit, Delete, Share URL (disabled)
1864	Fine crackles ver.23 (1min)	Lung Sound	Abnormal	Public	2023-07-06 18:22:07	Admin	Preview, Edit, Delete, Share URL (disabled)
1861	SAMPLE Case	Heart Sound	Abnormal	Public	2023-07-08 19:31:44	Admin	Preview, Edit, Delete, Share URL (active)

In that popup screen there are setting buttons for “Heatmap” and “Monitoring”. Turn ON if necessary (When monitoring is turned on, cases will be displayed on the user’s (kikuzosound.com) Live Monitoring screen.)

Administrator screen

The screenshot shows a 'URL Share' popup window. It contains a text field with a URL, a 'COPY URL' button, and several toggle switches. The 'Heatmap' and 'Monitoring' switches are highlighted with blue and red boxes respectively. There are also 'Name Input' and 'Password' fields with their own toggle switches.

URL: https://kikuzosound.com/ipax_url/1861?v=1687448423000&asset_ver=1687448423

Expiry Date: validity (2023/07/11) None

Heatmap:

Name Input:

Monitoring:

Name Input:

Password:

Save

User screen

The screenshot shows the user interface for Kikuzosound.com. The 'Live Monitoring' menu item is highlighted. The main content area displays a table of cases available for monitoring.


Label	Title	Monitor	Actions
ipax	SAMPLE Case	白岩 貴子	Enter
ipax	AS (Non pulse, 1min)	小坂貴子	Enter

We can also create our own content and case studies.

② Use selected cases for Live Monitoring

Select "Live Monitoring" from the left menu of the management screen to use. When users belonging to the same group open content on the dedicated site (kikuzosound.com), the screen is shared with the administrator. You can set the content to be published to users on the management screen.

Administrator screen

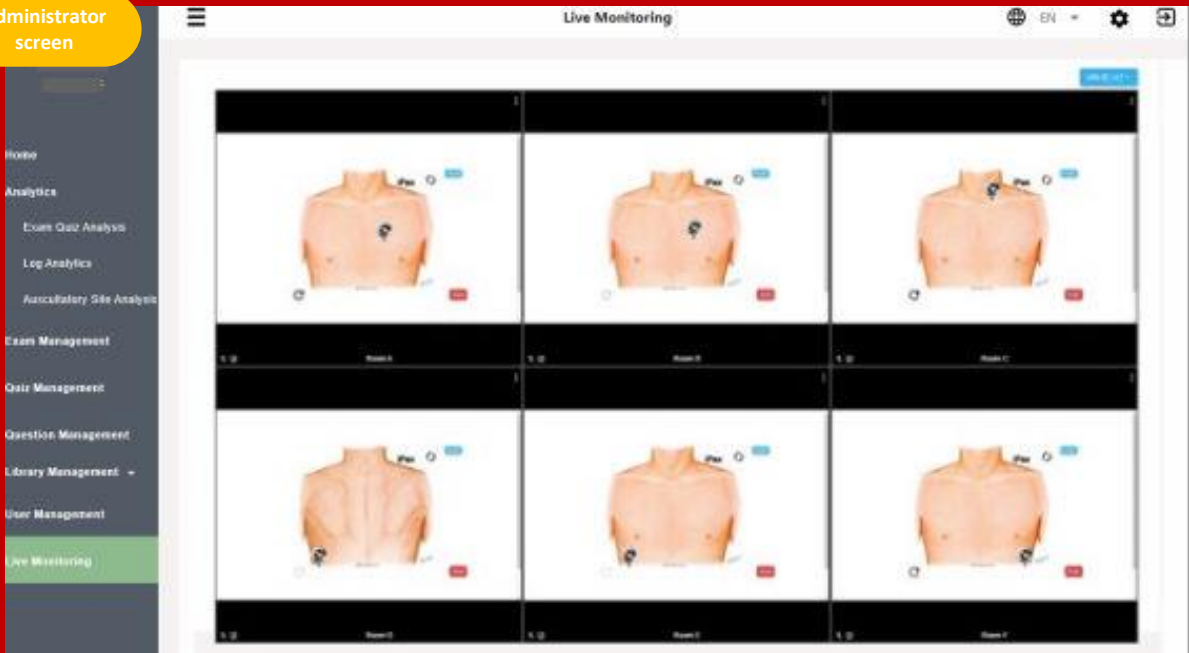
③ View the site auscultated by the user

After using iPax, press the "Finish" button at the bottom right of the screen to output the auscultated parts to the management screen.

User screen


Administrator screen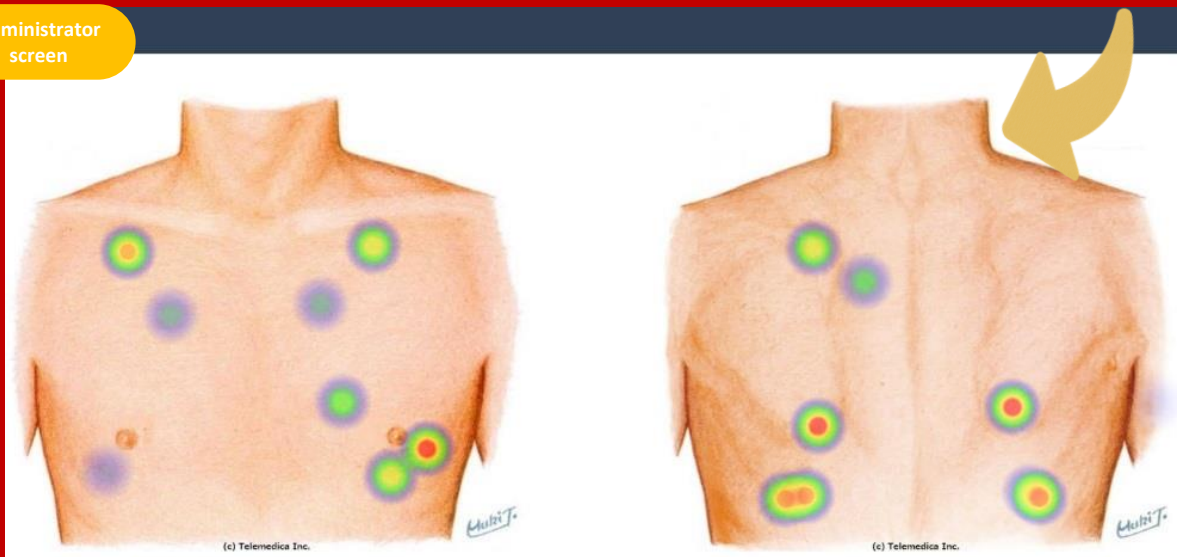

4 Manage Logs




On the management screen, you can view the usage time of the entire group, the average score of quizzes, and the history of each user. The data can be downloaded as CSV.

administrator screen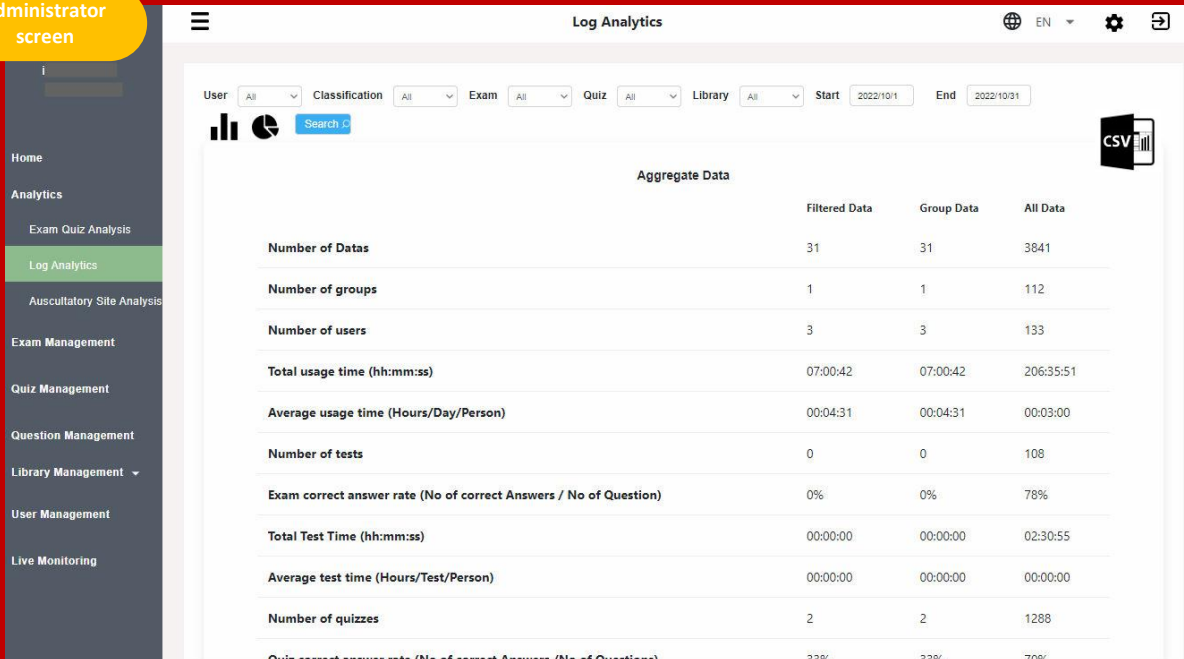


■ Many other features...

Images, videos, and audio files can be placed and distributed to users (students).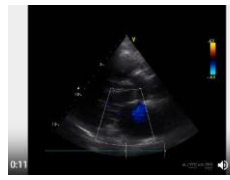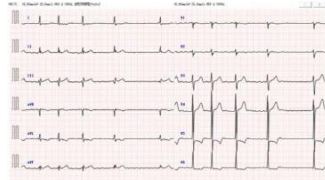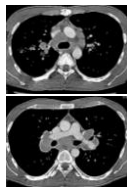

The chest and back illustration of the iPax auscultation can be changed to other illustrations and photos.

If you don't need auscultation, you can set it without auscultation.

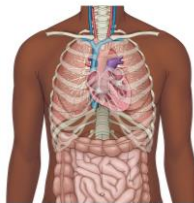⑤ Distribute basic subjects' content and case studies

You can freely add text, images, videos, etc.

You can distribute the content of basic subjects and publish information of case studies.

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(C) 2020 Telematica Inc.
In order to prove that it is the original sound, special signals are incorporated into this sounds.
Fund: KizukoSound.com

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Mr. Fonette
iPax admin

We may change the contents of this document as needed.
Please check the website for the latest information.

We believe that improved auscultation techniques lead to better diagnostic quality and patient satisfaction. We developed iPax to hone that technique. iPax has the ability to produce auscultation sounds that are close to the sounds of actual patients, and to change the sounds continuously. The only way to acquire auscultation skills is to listen to repeated sounds. iPax, which can be used on smartphones, will contribute to improving the quality of education in that regard.

If you need a specific auscultation sound, please contact Telemedica. We will create realistic heart and lung sound cases with our editing technology and provide them on iPax.

Telemedica Inc.
1-3-9, Aobadai, Aoba-ku, Yokohama
Email ask@telemedica.co.jp